## CURATEC HIDROGEL COM ALGINATO

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#### DESCRIÇÃO:

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### Instructions for use

#### DESCRIPTION:

CURATEC Hydrogel with Alginate is a gel consisting of purified water, propylene glycol, carbomer 940, triethanolamine, calcium and sodium alginate, preservatives and carboxymethylcellulose. The gel moistening the wound environment, facilitating scar healing and leading to autolytic debridement or easing mechanical debridement. This is a primary, non-sterile, transparent and viscous dressing.

#### ACTION MECHANISM

Together with the carboxymethylcellulose, the alginate retains water in its structure, acquiring a gel format. This allows them to help in exuding wounds by absorbing the exudation by maintaining its gel properties. Together with water and propylene glycol, the gel-like hydrated structure provides an environment that is ideal for the cicatrization process because it allows the mollification of possible unviable tissues which delay the process, easing autolytic debridement or its mechanical removal.

The wound hydration process provided by the hydrogel with alginate not only stimulates the skin healing process, it also alleviates pain due to the moistening of possible nerve endings exposed in the wound.

#### INDICATIONS:

It is indicated for treating dry, barely moist and medium exudation wounds with the presence of unviable tissue (necrosis and dliaceration), and also for stimulating granulation and epithelization by maintaining a moist environment. It can be applied in wounds of any etiology. CURATEC® Hydrogel with Alginate can be used under medial supervision in infected wounds. In the event there is a suspicion of infection during the use of the product, the possibility of using a joint systemic treatment should be considered.

These indications include several types of wounds, such as:

- Venous, arterial or pressure ulcers
- Second degree burns
- Abrasions
- Lacerations

#### CONTRAINDICATIONS:

The product shall not be used in surgical incisions and on patients with known sensitivity to the product or any of its components.

The application of hydrogel with alginate should not surpass the skin of the surrounding area of the wound.

In cases of irritation (allergic reaction) or any other adverse effect during the use of the product, it should be immediately discontinued and medical advice sought.

#### INSTRUCTIONS FOR USE

Cleanse wound very well with a 0.9% physiologic solution or according to established protocols at the medical institution

Dry only the periwound region

Apply the CURATEC® Hydrogel with Alginate directly on the wound bed - do not exceed the wound surrounding area

Close with a secondary cover and secure it

To remove the CURATEC® Hydrogel with Alginate, irrigate the wound bed very well with physiologic solution, if necessary

#### CHANGE INTERVALS

The CURATEC® Hydrogel with Alginate should be changed when the secondary dressing shows signs of saturation or when the dressing is removed for routine changes. The need of change frequency should be assessed by the health professional in charge.

The CURATEC® Hydrogel with Alginate can be left on the wound for up to 3 days. For infected wounds, change it at least every 24 hours.

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#### PRECAUTIONS:

The CURATEC® Hydrogel with Alginate is not recommended as the only means of treating the wound. Seek professional advice. During application, do not exceed the surrounding wound area.

For highly exuding wounds, the use of a more absorbent dressing is recommended, such as an alginate or a piece of foam.

The use of the product should be monitored by a health professional who should use surgical gloves when applying the product to maintain hygiene for both professional and patient.

Discard the primary package (tube) according to the hazardous hospital waste procedures determined by the medical facility.

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Do not use the product if the package is damaged.

#### STORING:

The product should be stored and transported at room temperature and protected from light and damp. It should be kept in its original, intact packaging.

#### EXPIRY DATE:

The 2-year expiry date is valid when the above-mentioned storage conditions are complied with. After opening, the product should be used within 28 days maximum.

#### AVAILABLE FORMATS:

The product is sold in individual aluminum tubes, packed in sealed cartridges or kits with a safety-seal and instructions for use.

The product is sold in the following available formats:

- 5g
- 10g
- 15g
- 20g
- 25g
- 30g
- 40g

Hydrogel with Alginate tubes of 20g, 25g and 30g;

The shipping cartons come with 6 cartridges of 3 tubes each.

Hydrogel with Alginate tubes of 5g, 10g, 15g, 20g, 25g, 30g, 40g, 50g, 60g, 70g, 80g, 85g and 100g;

The shipping cartons come with 12 cartridges of 1 tube each.

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